

Corival cpet

Can be controlled by all known Cardio Pulmonary Exercise devices



Highlights

Reliable and reproducible stress tests

The experience of professionals who calibrate many ergometers showed that the Lode ergometers are the most reliable across the complete workload and rpm range and still within specifications even after many years of intensive use.

High standards

Lode is a socially and environmentally responsible company. All Lode products are RoHS/WEE compliant and Lode is ISO 9001:2003, ISO 13485:2008 and FDA 510K certified. All medical products comply to MDD 93/42/EEC, incl. IEC 60601-1.

Various test modes

Besides the hyperbolic (rpm-independent) mode that is used most of the time, the standard control unit offers several other test modes, like the fixed torque mode and the linear mode. These modes can be used in both manual and terminal mode.

Q-factor equal to road-bike

The Q-factor of the ergometer is equal to the Q-factor of road bikes, creating perfect training circumstances.

Rotatable handlebar with new lever

The new designed lever makes it even easier to adjust the handlebar. The handlebar can be rotated 360 degrees and is constructed in such a way that the test subject can be installed comfortably at every seating height.



Corival cpet

Can be controlled by all known Cardio Pulmonary Exercise devices



The Corival is one of the most popular ergometers worldwide. The low start-up load of 7 Watt is first-class. The Corival cpet is standard supplied with a communication module and can therefore be easily controlled by all known stress ECG and pulmonary devices in the world. The workload, rpm and time can be readout from the 3,5" colour display. The Corival has an eddy current electro-magnetic braking mechanism. The biggest advantage of this system is the accuracy which is one of the most important Lode principles. With this ergometer, the stress tests performed are reliable and reproducible. The workload is adjustable in a range of 7 to 1000 watt. The ultralow step-through enables easy access to the ergometer and the latest design guarantees a perfect ergonomic position. Moreover, the noise level is reduced to a minimum.

A USB A-B cable for service purposes as well as connecting to ECG and pulmonary testing devices will be standard delivered with the product. To connect older ECG and pulmonary testing devices with RS232 or other connectors you need a special interface cable that can be ordered separately.

Features



Compatible with ECG and pulmonary devices

The Lode ergometers have both analog and digital interfaces and can be controlled easily by all known stress ECG and pulmonary devices available in the world. This is one of the reasons why the Lode ergometers are very popular worldwide.

**7
watt**

Extreme low start up load

The extreme low start-up load of 7 watts and the adjustability in small steps of 1 watt make this ergometer perfectly suitable for many different applications. The standard control unit shows multiple ergometry parameters and you can determine your specific default setting and start-up menu.



Low noise

Due to accurate manufacturing and the careful choice of materials the product has an extremely low noise level.



Accurate over a long period of time

The Lode ergometers are supplied with an electro-magnetic braking mechanism of Lanooy (eddy current). The biggest advantage of this braking system compared to a friction braking system is the absolute accuracy and the accuracy over time. Moreover, friction braking systems have more wearing parts.



RS232 connectivity

RS232 ports enable connectivity to most ECG and ergospirometry devices as well as PC's.



Readout out of saddle height

The height of the saddle is stepless adjustable and can be read-out on the saddle shaft



Perfect ergonomic position

Improved ergonomic position according to the latest requirements.



Ultra-low step-through

The lowest possible step-through guarantees easy access to the ergometer for all test subjects: a must for people who are not so mobile!



Hidden connectors

The cables are connected to the ergometer under the ergometer, which means that the test subject or operator cannot bump onto the connector.



USB connectivity

USB to connect to PC or ECG or ergospirometry products facilitates easy connectivity.








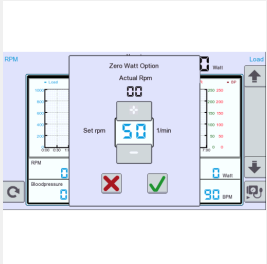





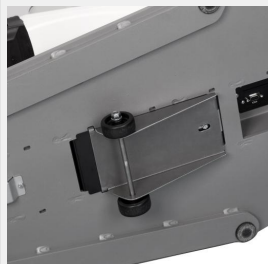



THE STANDARD IN ERGOMETRY

Corival cpet

Can be controlled by all known Cardio Pulmonary Exercise devices



Corival cpet can a.o be extended with the following options:

<p>Control Unit with touch screen 7" for ergometer Multifunctionality</p>  <p>Partnumber: 945834</p>	<p>Control Unit with touch screen 7" - ordered additionally Multifunctionality</p>  <p>Partnumber: P945834</p>	<p>Programmable Control Unit with 7" Touchscreen Programmable</p>  <p>Partnumber: 945835</p>	<p>Blood Pressure Measurement for bicycle ergometer Accurate and reliable</p>  <p>Partnumber: 945825</p>	<p>Electric adjustable saddle height Easy and accurate positioning</p>  <p>Partnumber: 960810</p>
<p>0-Watt start-up system Lowest possible startup power</p>  <p>Partnumber: 960805</p>	<p>Heart rate for bicycle ergometers Heart rate in beats per minute</p>  <p>Partnumber: 945821</p>	<p>Adjustable cranks Optimal force application</p>  <p>Partnumber: 928804</p>	<p>Shortened saddle shaft Increase flexibility for smaller people</p>  <p>Partnumber: 960806</p>	<p>Saddle for children Versatile ergometry</p>  <p>Partnumber: 401068</p>
<p>Saddle extra large Versatile ergometry</p>  <p>Partnumber: 401080</p>	<p>Transportwheel for Corival Easy transportation indoors</p>  <p>Partnumber: 960801</p>	<p>Network Module Easy networking with LEM and LCRM</p>  <p>Partnumber: 945851</p>	<p>RS232 cable Easy connection</p>  <p>Partnumber: 930911</p>	<p>USB to Serial cable Easy connection</p>  <p>Partnumber: 226012</p>

Corival cpet

Can be controlled by all known Cardio Pulmonary Exercise devices



Specifications

Workload

Minimum load	7 W	
Maximum peak load	1000 W	
Minimum load increments	1 W	
Maximum continuous load	750 W	
Hyperbolic workload control	✓	
Linear workload control	✓	
Fixed torque workload control	✓	
Maximum rpm independent constant load	150 rpm	
Minimum rpm independent constant load	30 rpm	
Optional heart rate controlled workload	✓	
Electromagnetic "eddy current" braking system	✓	
Dynamic calibration	✓	
Power range at maximum rpm (maximum)	1000 W	

Accuracy

Workload accuracy below 100 W	3 W	
Workload accuracy from 100 to 500 W	3 %	
Workload accuracy from 500 to 1000 W	5 %	

Comfort

Q-factor	180 mm	
Allowed user weight	180 kg	396.8 lbs
Handlebar adjustment angle	360 °	
Adjustability range seat	300 mm	11.8 inch
Minimum leg length user	645 mm	25.4 inch

User Interface

English user interface	✓	
Norwegian user interface	✓	
Czech user interface	✓	
Danish user interface	✓	
Dutch user interface	✓	
Finnish user interface	✓	
French user interface	✓	
German user interface	✓	
Italian user interface	✓	
Japanese user interface	✓	
Korean user interface	✓	
Polish user interface	✓	
Portuguese user interface	✓	
Russian user interface	✓	
Spanish user interface	✓	
Turkish user interface	✓	
Ukrainian user interface	✓	
Readout RPM	✓	
Readout Time	✓	
Readout Power	✓	
Set Resistance	✓	
Terminal operation mode	✓	
Screen size (diagonal)	8.9 cm	3.5 inch
Touchscreen	✓	

Connectivity

Lode 38K4 interface protocol	✓	
Lode interface protocol	✓	
Lode WLP interface protocol	✓	
Ergoline P10 interface protocol	✓	
Ergoline P4 interface protocol	✓	
Schiller interface protocol	✓	
Bosch EKG 506 DS interface protocol	✓	
USB connector	✓	
RS232 in connector	✓	

Dimensions

Product length (cm)	105 cm	41.3 inch
Product width (cm)	46 cm	18.1 inch
Product height	114 cm	44.9 inch
Product weight	65 kg	143.3 lbs



THE STANDARD IN ERGOMETRY

Corival cpet

Can be controlled by all known Cardio Pulmonary Exercise devices



Power requirements

Power cord length	250 cm	98.4 inch
100 - 240 V 50/60 Hz (160 Watt)	✓	

Standards & Safety

ISO 13485:2003 compliant	✓
ISO 9001:2008 compliant	✓
IEC 60601-1:2005	✓

Certification

CE class Im according to MDD93/42/EEC	✓
CTüVus according to NRTL	✓
CB according to IECEE CB	✓

Order info

Partnumber	960900
------------	--------

**Specifications are subject to change without notice.*